
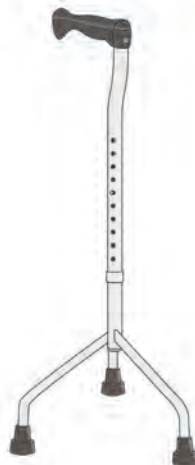
**TRIKOJĖ IR KETURKOJĖ LAZDĖLĖ
TRĪSKĀJU UN ČETRĶĀJU SPIEKIS
KOLMEHARULINE KEPP JA NELJAHARULINE KEPP
TRIPOD AND QUAD CANE
ТРОСТЬ 3-Х и 4-Х ОПОРНАЯ
TRINOŽNA IN ŠTIRINOŽNA PALICA
HŮL S TROJNOŽKOU A ČTYRNOŽKOU**

Naudojimo ir priežiūros instrukcija
Lietošanas un kopšanas instrukcija
Kasutusjuhend
User manual
Руководство по эксплуатации
Priročnik za uporabnike
Uživatelská příručka

01-7782C

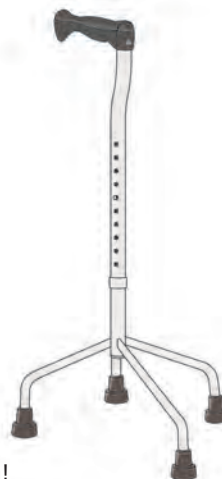01-2620

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Observe instructions for use! Please read these instructions carefully before use and keep it for later reference!

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EN Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

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EN A walking stick is made for people who have mobility problems. The height of 01-7782C and 01-2620 stick is adjustable. In order to change the height of the walking stick, press a button in the middle of the stick. The same button helps to change the position of the handle. In this way the cane is suitable for both handles.

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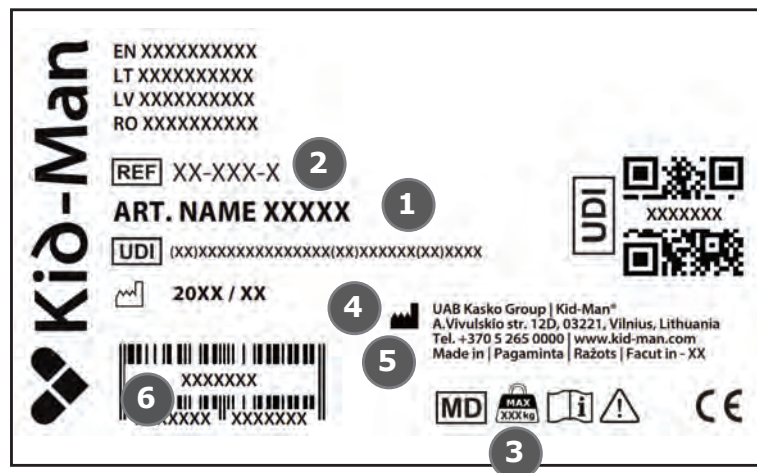
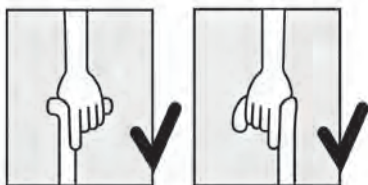
EN To adjust the length of the stick, put the pin situated in the top of cane and slide the top section up or down setting the comfortable height, then pop the pin back in appropriate hole. Hold your stick with your index finger point down the length of stick for optimum weight distribution.

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EN It is recommended to hold the stick in the opposite hand of injured leg. However, if cane is used only as walkind aid, you may hold it the hand more comfortable for you. Always make sure two points are kept on the ground to ensure safety.

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EN Please wipe your cane periodically with soupy warm water. Check rubber tip regularly, replace if necessary.

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EN Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group” after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group”, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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