MUVI100

VAIKŠTYNĖ SU STALIUKU
STAIGULIS AR GALDIŲU
KÄIMISTUGIRAAM
WALKER WITH A TABLE
ХОДУНКИ СО СТОЛИКОМ
CADRU DE MERS CU MASĂ
CHODÍTKO S OPĚRNOU DESKOU
HODALICA SA STOLÍCEM
HODULJA Z MIZO
CHODÍTKO SO STOLÍKOM

- i**
- LT** Naudojimo instrukcijos
 - LV** Lietošanas instrukcija
 - EE** Kasutusjuhend
 - EN** User manual
 - RU** Руководство по эксплуатации
 - RO** Manual de utilizare
 - CZ** Návod k použití
 - HR** Korisnički priručnik
 - SL** Uporabniški priročnik
 - SK** Návod na použitie

- LT** Prieš naudodami atidžiai perskaitykite naudojimo ir priežiūros instrukciją!
- LV** Pirms lietojiet, lūdzu izlasiet uzmanīgi lietošanas un kopšanas instrukciju!
- EE** Enne kasutamist lugege hoolikalt kasutusjuhendit!
- EN** Observe instructions for use! Please read these instructions carefully before use and keep it for later reference!
- RU** Прежде чем использовать внимательно прочтите инструкцию!
- RO** Vă rugăm să citiți cu atenție aceste instrucțiuni înainte de utilizare și să le păstrați pentru referințe ulterioare!
- CZ** Dodržujte návod k použití! Před použitím si pozorně přečtěte tyto pokyny a uschovejte je pro pozdější použití!
- HR** Prije upotrebe pažljivo pročitajte ove upute i sačuvajte ih za buduću upotrebu!
- SL** Pred uporabo natančno preberite ta navodila in jih shranite za poznejšo uporabo!
- SK** Dodržujte návod na použitie! Pred použitím si pozorne prečítajte tento návod a uschovajte si ho na neskoršie použitie!

PRATARMĖ | IEVADS | EESSÕNA | PREFACE | ПРЕДИСЛОВИЕ | PREFATA | PŘEDMLUVA | UVOD | PREDGOVOR | PREDHOVOR

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EN The walker should be stored in a dry environment, away from direct sunlight. When in storage the walker can be folded up. The walker should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

Use warm soap water to clean metal parts. Do not use aggressive detergents or acids.

Regularly check adjustable walkers parts: brakes, knobs. Make sure they are working properly.

Regularly check the walkers table material.

**PRIEŽIŪRA | UZKOPŠANA | HOOLDUS JA KORRASHOID | CARE AND MAINTENANCE | УХОД
ÎNGRIJIRE ȘI MENTENANȚĂ | ÚDRŽBA A PÉČE | ODRŽAVANJE I NJEGA | NEGA IN VZDRŽEVANJE | ÚDRŽBA A STAROSTLIVOST'**

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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